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Instruction

Safety instructions:

To maintain your guarantee please pay attention to the following tips and observe the following manufacturers requirements and restrictions. Please also pay attention to the safety instructions of Einhell, regarding their Einhell-Battery heat gun „TE-HA 18 Li – Solo“ (Item. No. 4520500).

The oxalic-acid-vaporiser-attachment (Rohr-Oxalsäure-Verdampfer-Vorsatz) for Einhell-Battery heat gun is designed only for using on medical treatment purpose of beehives, by using oxalic acid. Be aware of your local rights and restrictions, using drugs and medication for your beehive, in particular treading oxalic acid to your hive.

Like any other acid, oxalic acid must be stored on a safe place, inaccessible for children. The acid causes burns and is harmful to health by touching or swallowing. Do not breathe acid steam or dust. In case acid gets into your eye, thoroughly rinse it with water and consult a doctor immediately. Always wear acid protective safety clothing's such as eye protection goggles, acid-resistant gloves ([Item-no. 11465](#)) and FFP 3 mask ([Item-no. 11587](#)). Avoid splashes when handling with acid and be sure to provide a bucket of water especially if there is no access to fresh tap water at your place.

You should only set and use Einhell-Battery heat gun at level one. Only use level two for temporary preheating, this way you prevent overheating the heat gun. Always hold the heat gun upright with its installed vaporizer attachment, the lid of the attachment only rests on the front-end assembly.

The whole vaporizer attachment including its parts gets very hot while using. In addition, when turning of the heat gun, the attachment still radiates harmful warmth (risk of burns), so do not touch until the attachment cooled down properly.

Do not place flammable or thermally sensitive objects close to the hot vaporiser attachment or parts of it. Prevent touching hot parts at any time and be sure that you only place the hot vaporiser attachment on a heat-resistant surface. Supervise the cooling down process at any time.

By using the vaporiser attachment at a bee house, treatment need to be conducted from the outside. While or after applicate, make sure bee house is aerated. Nobody should stay inside bee house while treading the bees. Generally make sure no unprotected or uninvolved person reside into the danger zone. Pay attention to wind and schedule your work in consideration of winds direction.

To carry out a proper treatment with the vaporiser attachment, also follow the next advice carefully:

Bee's treatment should only be done by qualified beekeepers. Always check the disease burden by Varroa mites on each of your hive, before medication. You should never treat your hives without anamnesis.

For understanding your hives burden of Varroa mites, we recommend our pest control drawer ([Item-no. 10241](#)), if you are using bottom boards with ventilation. If you are using closed bottom boards without ventilation we suggest using our diagnosis board ([Item-no. 11564](#)). However, Varroa-Tester ([Item-no. 11582](#)) also provides a great way to check Varroa pressure on your hive, no matter what kind of hive or bottom board you have.

Do not treat your hives during or just around honey production. Always use the right amount of medication. Do not overdose oxalic acid supplement, it lead the bees to get very restless. Furthermore young brood (young larva) or bees ready for hatching react very sensible suspended on too much acid. We highly recommend not to tread one bee generation per hive twice with oxalic acid, it causes acidification on bees and let them die.

Should your bees ever show deep concern about your treatment of oxalic acid (nymphs fall down, strong bee ventilation, bees swelling out of the hive), instantly stop treatment or reduce the doses of medication. Perhaps it's necessary to contact a beekeeper expert.

Anyway, historically evaporating oxalic acid has been a bee safe and helpful way to treat your hive against mites. Problems by using this way of treatment method occur very rarely or just if malpractice took place. Never the less, because of individual factors from one user to another or from one treatment to another, there is no guarantee that danger or problems with this product appear, even though you followed all safety instructions stated.

The manufacturer rejects all claims and any liability in terms of a correct Varroa treatment with the vaporiser attachment for Einhell-Battery heat gun. Even more, the manufacturer is not liable for any damage, traced back to the application of vaporizer attachment or any other act given in this instruction.

Specification / How to use: Oxalic acid vaporiser attachment is designed for Einhell-Battery heat gun „TE-HA 18 Li – Solo“ (Item-no. 4520500). It also fits following devices: Dewalt Akku-Heißluftpistole 18V DCE530, Milwaukee Akku-Heißluftgebläse M18 BHG. Not compatible by using Bosch 18V GHG und Rapid RX1000. With the help of this attachment it's possible to evaporate oxalic acid for Varroa mite control very easy, using the air flow of heat gun to blow the oxalic acid smoke into the beehive. Depending on battery capacity, the Einhell heat gun (accessible separately) prevents enough energy for treading up to 10 beehives with one charge. Without any powerline needed mobile and fully independent of power supply. It's intended to get the acid smoke into the hive via entrance or over a self-made whole in your bee box.

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Assembling: Please assemble the attachment like it's given on the picture to the right. Align the fill port upright at the front of Einhell heat gun. Secure the attachment additional with the hose clamp (included in the package). Make sure the attachment sits tight to Einhell heat gun before heating. In case some parts or the whole attachment gets loose, first make sure the assembly cooled down properly before fasten the parts again (risk of burns).

Using: Always wear proper safety clothing's (see above). Prepare a bucket of cold water to cool down the copper pan later. To start, first turn on the heat gun at level 2 for around 2-3min (preheating). Now turn off the device and place the copper pan, which you already filled with oxalic acid, into the heating chamber by using a needle nose pliers. **Attention! Everything gets hot – Only touch the heating chamber cap knob wearing suitable heat protective gloves!** Close the heat chamber again. You are now able to turn on heat gun at level 1. As soon as oxalic acid smokes evaporates out of the attachment, you are able to start medicating your hives.

Date of Application: Oxalic acid is applied to brood less beehives or brood less artificial/natural swarms. The outside temperature shouldn't be more than +3°C, because winters bee cluster or the bees itself inside their hive must linger a bit loose. Facing a natural mortality of more than 0,5 Varroa mites per day in wintertime (e.g. in Germany at November/December, varies according to your residence), it's necessary to treat your bees once. To get the treatment done right, please pay attention to the instructions above.

Treatment: Insert the plastic hose into the entrance of your hive or into a separately drilled hole. Furthermore it's necessary to close hives entrance by using a foam-plastic strip ([Item-no. 10640](#)) or perhaps a cloth, impregnated with a little bit of acetic acid. To get the bees unblocking the entrance for treatment. Ventilation grits or other openings need to be closed tidily. After about 30-45 seconds, the beehive will be filled with smoke and white clouds of smoke will curl off of the hive. Now turn off heat gun and quickly switch to another hive to turn on the heat gun again. Keep the entrance closed for a few more minutes after treatment. It's very helpful to place the impregnated acetic acid cloth at the entrances of your hives before you start the actual treatment. This way you make sure the bees are exposing the entrance and with the cloth you immediately got a tool to close hive entrance right after treatment. This way prepared, you will only need a few seconds for each hive, getting the smoke inside.

As soon as saturated smoke gets out of every gap of your hive or the entrance, you are able to move on to the next hive. Depending on battery capacity, you're able to treat up to 20 hives. One filling oxalic acid of the copper pan last for up to 10 hives. One treatment usually only takes about 30-45 seconds (depending on hives volume). **We highly dissuade to treat one bee generation per hive twice with oxalic acid, the bees will acidify and die.**

End of treatment: Once you finished treatment or if there is no more smoke coming out of your hive, turn off heat gun. Only open or loose the attachment if it cooled down to room temperature. The copper pan you either throw into the water bucket, by turning around the heat gun with its installed attachment, or you carefully get out the copper pan by using the needle nose pliers again. Refill the copper pan with oxalic acid in case you want to continue the treatment process. Otherwise you turn right and pull out the attachment of Einhell heat gun. To get the treatment of more than 10 hives done even faster, we recommend to buy a separate copper pan ([Item-no. 11589](#)). Prepare this spare pan before treating so you are able to continue the treatment right after one per-charge of acid has been used of. By using a new copper pan filled with acid, continue treatment as long as you treated every colony. Once you are done, the attachment needs to cool down completely before removing and cleaning.

Maintenance / Care: Always disconnect the attachment of Einhell heat gun for maintenance or caring purpose. Frequently check the cooled down outlet tube, it tends to plug if you don't preheat. Always make sure the outlet tube is clear. After using, the heating chamber and its lid need to be washed thoroughly with water. Wear acid-safe gloves ([Item-no.11465](#)). To prevent contact corrosion, dry the lid separately of heating chamber. If you need to replace the transparent tube ([Item-no. 11592](#)) of the oxalic acid vaporiser attachment, visit our website to place an order.

Disposal / Recycling:

Hand over retired devices properly to a public collecting point or send them back to manufactures address.

Guaranty claim:

The manufacturer provides a guaranty of two years, beginning at selling- or billing date of retailer or manufacturer. Original invoice documents are necessary. Only repairs conducted by manufacturer are permit.

Technical specifications: **Weight:** approx. 0,262 kg | with hose clamp approx. 0,280 kg **Size (W x D x H):** ca. 3,5 cm x 23 cm x 8 cm
Cap diameter of attachment: approx. 3,7 cm

The manufacturer rejects all claims and any liability, which connect to a disregarded use; operations in breach or not explained instructions out of this guide. It's forbidden to reproduce or copy this instruction guide or parts of it (Copyright). Contraventions will be prosecuted at any time.

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Bienen-Voigt und Warnholz GmbH & Co. KG